


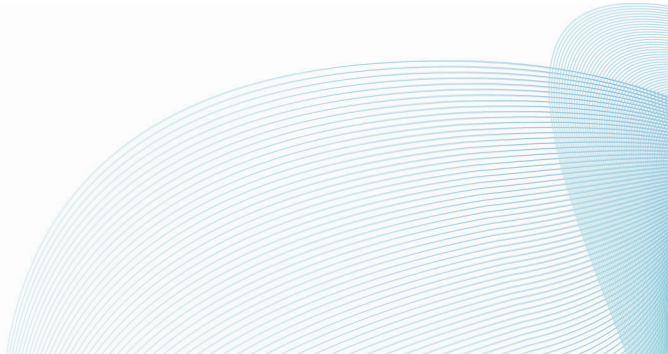

Boot Extender Operating Manual

For use with Formthotics Heating Machine V20-100/V20-200



IMPORTANT SAFETY INSTRUCTIONS

BEFORE USING THIS PRODUCT READ ALL INSTRUCTIONS AND CAUTIONARY MARKINGS IN THIS MANUAL AND ON THE APPLIANCE

When using this accessory with an electrical appliance, basic precautions should always be followed, including the following:

WARNING

- 1 Use these boot extenders only in conjunction with the Formthotics Heating Machine V20. This product is not compatible with any other device.
 - 2 The boot extender is hot when in use and for a period after use. To avoid burns, do not let bare skin touch hot surfaces.
 - 3 Read all warnings and operation instructions included with the Formthotics Heating Machine V20 before fitting this product.
 - 4 Operate the Formthotics Heating Machine V20 fitted with boot extenders only on a stable and level surface.
-

ABOUT THE BOOT EXTENDER

The boot extender for the Formthotics Heating Machine V20 is engineered to meet the highest standards of quality and functionality.

It is specifically designed for use with ski boots, snowboard boots or other fully sealed footwear. When used in conjunction with the Formthotics Heating Machine V20, it gently heats any Formthotics orthoses to the optimum temperature for custom moulding to the foot and footwear. It has only one setting and is very easy to use.

OPERATION

STEP 1

Loosen the fastening system on the boot. Remove any existing insoles and insert the Formthotics into the boots.


Insert a boot extender into one of the boots in the orientation shown until the end of the boot extender touches the Formthotic. Pull the boot extender back by roughly 1cm.

Tighten the fastening system on the boot around the boot extender. Repeat with the other boot.



STEP 2

Fit the boot and boot extender onto the outlets of the Formthotics Heating Machine V20 as shown. The weight of the boot will ensure the boot extender fits securely onto the outlet.

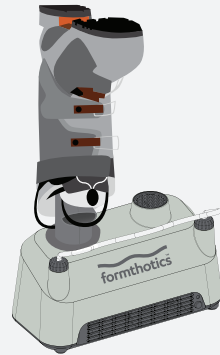

STEP 3

Press the Start button. The LED indicator will change from green to blue to indicate the heating cycle has started. After 3 minutes, the LED indicator will begin to flash green and a buzzer will sound. Push the button twice to restart the heating cycle.

Do not leave the machine unattended while in operation.

STEP 4

Remove boots and loosen fastening system to allow removal of the boot extender.

Note: The exhaust outlet and inserted end of the boot extender will be hot. Remove the boot extender by handling the bottom of the extender below the outlet.

Fit the boots to the customer immediately and have them stand in them 2-3 minutes to allow Formthotics to mould to the shape of the foot and shoe. Turn the power switch off.

This machine is intended for heating Formthotics products only. Do not use the machine for any other purpose, or any other brand of orthotic or insole.

WARRANTY & SERVICE POLICY

Foot Science International warrants this product against any defects due to faulty material or workmanship for a two-year period from the date of purchase.

The warranty does not include damage resulting from misuse. If the product becomes defective during this period, we will repair or replace it free of charge. This product is not user serviceable, please return the unit to place of supply for service.

Disposal: Please consider recycling or returning this product to Foot Science International for dismantling at the end of its useful life.



Foot Science International Ltd
26 Dakota Crescent, Sockburn
Christchurch 8042, New Zealand

Phone +64 3 348 2115

Email info@formthotics.com

Visit formthotics.com